


Face Mask Pattern

(version 2)

<http://www.craftpassion.com/?p=26304>

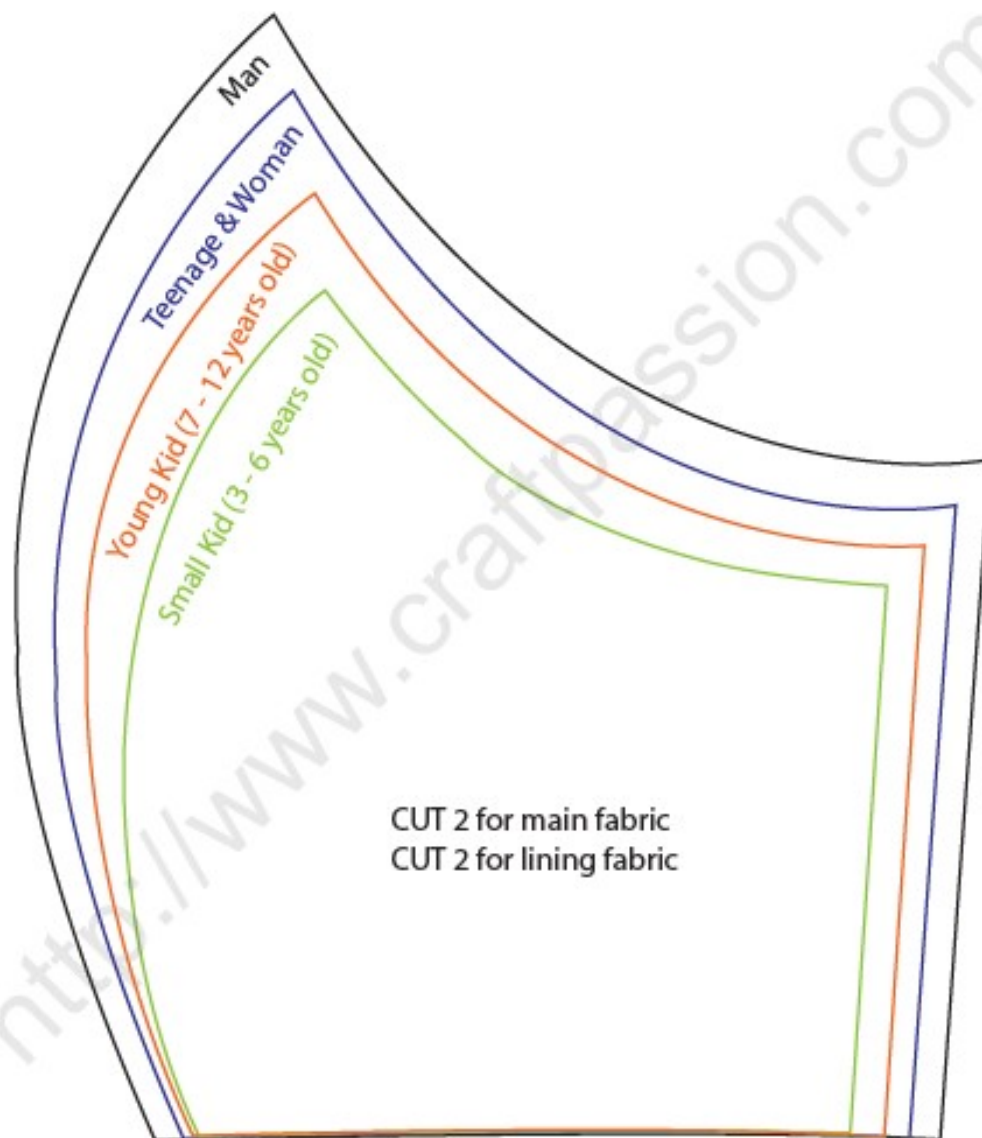

2"

Please check your scale here after print

Note: Please add 1/4" seam allowance for all the pattern

Copyright © craftpassion.com
Meant for hobby and personal use only. Do not sell.

Face Mask Sewing Pattern

Courtesy of : <https://www.craftpassion.com/wp-json/mv-create/v1/creations/19/print>

Materials

- Fabric—100% cotton with high thread count:
- Main Fabric: 13" x 7", prewash
- Lining Fabric: 12" x 7", prewash
- Elastic cord for ear loops 16", or, shoe lace/ribbon/cord with at least 44" length for head tie

Directions for Face Mask With A Pocket For Filter Insert

Get ready for all the materials as stated in the material list above.

Fold the fabric with the wrong side facing each other, place the paper template on it, pin.

Cut the fabric out with $\frac{3}{8}$ " seam allowance, except the side of the face mask: 1" for the outer layer, $\frac{1}{2}$ " for the inner layer.

Trace the sewing lines with the tracing paper and tracing wheel.



Turn the pieces over and pin the pair together with the right side facing each other. Sew the curved lines as shown in the picture.

Draw a line $\frac{1}{4}$ " from the original side sewing line of the inner layer, do the same to the other side of the inner layer.

Clip the seam allowance about $\frac{1}{2}$ " apart. This will let the fabric's raw edge to stretch without pulling the seam when you turn it right side out. Repeat this on both the outer layer and the inner layer.



Turn the pieces right side out, press the seam allowance to one side, top-stitch near the seam line. Repeat the same to the inner layer.



Fold the side at the second line drawn next to the original sewing line of the inner layer. Fold the raw edge and hide inside the fold. Topstitch to sew it on. Repeat on both sides of the inner layer.

Place the inner layer on top of the outer layer, with the right side facing each other.

Sew the top and bottom seam lines.

You may notice that the edge of the inner layer is now $\frac{1}{4}$ " away from the sewing line of the outer layer. If it is not the case, please check if you sewing it correctly in the previous step.

Turn the face mask right side out. Press seam flat.

Double fold the top and bottom raw edges of the outer layer (both sides). Topstitch near the edge as shown in the picture. Repeat the same to the bottom seam line.

To make the pocket for the elastic band, fold the side raw edge $\frac{1}{4}$ " from the edge, then fold again so that the edge meets the edge of the inner layer.

Topstitch a vertical line to sew it on. The top and bottom have an opening for you to insert your elastic band.

This is the opening for the filter insert.

Add elastic bands to the pocket face mask the way you like and it is really to be worn.

Insert a surgical mask if you are using the face mask as a cover.

Insert a non-woven fabric if you can't find any surgical mask in the market.

